

# Phonak


## Fast Facts

### Tap Control


#### What is it?

Tap Control is a feature that allows controlling the hearing aid by a double tap instead of using the hearing aid button or a remote control.

1. Pause/resume streaming from many Bluetooth devices

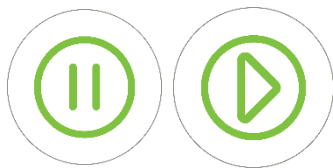

2. Accept/end phone call

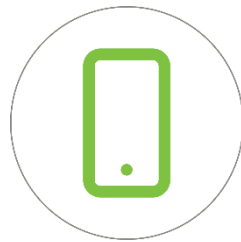

3. Start/stop voice assistant; Siri, Google etc



#### How to use it?

##### Clinician

- On by default in Audéo P-R and Audéo P-RT (performance levels 90 and 70)
- Personalization in Phonak Target > Device options > Manual controls
- Functions can be deactivated or assigned to the preferred ear
- A 'Double tap training' option, in Phonak Target, is available to ensure confidence in using the feature during their daily life

##### Client

- Activated by a double tap on the upper rear of the pinna or on the glasses
- Configurable by the client in myPhonak 4.0 app

#### What is the benefit?

Tap Control gives the client easy and convenient access to their streaming inputs without further complicating the functionality of the multi-function button. Twice as many participants preferred Tap Control over existing controls<sup>1</sup>.

1. Adler, M., & Woodward, J. (2020). New Tap Control: providing a preferred and reliable alternative to access direct streaming functions. Phonak Field Study News. Retrieved from [www.phonakpro.com/evidence](http://www.phonakpro.com/evidence). Accessed August 2020