

Phonak Cleaning wipes.

The cleaning wipe is a cleaning product for the maintenance of hearing aids and earmolds. It removes earwax responsible for the decrease in sound quality, increases the life of hearing systems and preserves their effectiveness. The cleaning wipe is presented in an individual pack of 60x80mm and are packed in cases of 30 units.

Using advice

- Open the bag and remove the wipe.
- Clean hearing aids or earmoulds with a wipe.
- Wipe the hearing aid with a soft, dry cloth.

It is recommended to clean hearing aids and earmolds every day.
The cleaning wipe is for single use. Discard the cleaning wipe after use.

Precautionary statements

Keep out of reach of children. External use.



Product composition

- Water
- Alcohol
- Caprylyl Glycol
- Ethylhexylglycerin
- Benzalkonium Chloride

029-1352-48/V1.00/2025-05/jb © 2025 Sonova AG. All rights reserved.